





SIRIS THUMB MCP SPLINT (MCP)

Hyperextension of MCP Joint with Active Flexion at the MCP and Abduction at the CMC Joint


- 1 Place a dot half way between the CMC and the MCP joints.

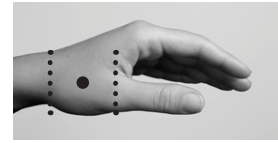

- 2 For length of proximal ring, choose an EZ-Sizer that sits in the MP volar crease and, when angled back, clears the dot.

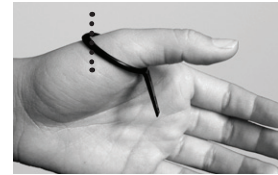

- 3 For width of proximal ring, choose a smaller EZ-Sizer that, when angled back, touches side to side along the thenar eminence and web space. This EZ-Sizer is not intended to clear or touch the "dot" as in step 2.

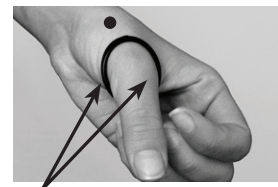

Touches side to side

- 4 For distal ring size choose an EZ-Sizer that, when rotated on its side, slides over the IP joint and reaches the web space.



- 5 The SN Holder can be put on after the two EZ-Sizers are in place to "visualize" the complete "splint". You cannot slide the "splint" on or off with the SN Holder in place. This is a "visualization" step only.

MCP OPTIONS:

(a) **Proximal Volar Extension (PVX)** is used to stabilize under the head of the metacarpal joint to reduce hyperextension. No measurement is needed. PVX can be manually adjusted to fit the curvature of the palm. Use when there is more than 20° hyperextension.

(b) **Bracelet** is used to keep the splint from sliding distal with normal use. No measurement is needed. The length of the bracelet may be adjusted for proper fit once the size of the splint is correct.

(c) **Proximal Radial Extension** and (d) **Ulnar Lateral Support** can be added to correct MCP deviation once the fit of the splint is correct.

'S' Hook can replace the bracelet lobster claw clasp for people with fine motor limitations.



SIRIS™ MEASURING INSTRUCTIONS

SILVER RING SPLINT COMPANY | PHONE: 434.971.4052 or 800.311.7028 | FAX: 434.971.8828 or 888.456.8828

CustomerService@SilverRingSplint.com | www.SilverRingSplint.com