





STABLE THUMB SPLINT (ST) WITH PVX

Hyperextension of MCP Joint with little or no Abduction at the CMC Joint

- 1** Choose the smallest Round Ringsizer that is snug when it slides over the IP joint.

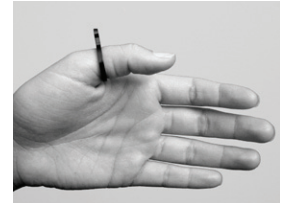

- 2** A bracelet is included with the splint. The length can be adjusted by a local jeweler or returned to us for adjustments.

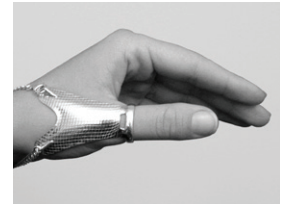

- 3** Proximal Volar Extension (PVX) is used to stabilize under the head of the metacarpal joint to reduce hyperextension. No measurement is needed. PVX can be manually adjusted to fit the curvature of the palm.



STABLE THUMB OPTIONS:

'S' Hook can replace the bracelet lobster claw clasp for people with fine motor limitations.

SIRIS™ MEASURING INSTRUCTIONS

SILVER RING SPLINT COMPANY | PHONE: 434.971.4052 or 800.311.7028 | FAX: 434.971.8828 or 888.456.8828

CustomerService@SilverRingSplint.com | www.SilverRingSplint.com