

SIRIS™ Swan Neck Splint & SIRIS™ Boutonniere Splint Worn in tandem.

The Silver Ring Splint Company encourages the use of two separate splints rather than one combination splint supporting both joints.

Two splints typically allow more normal hand function and flexibility. One or both splints can be used at a time depending on need. The rings are parallel to each other which allow them to fit on the same finger.

Some notes about deciding on correct sizing:

- 1) The two rings that almost touch should be about the same size {see picture}.
- 2) It is sometimes important to shorten the Boutonniere Splint by bending the proximal ring to a more perpendicular position or in other words to around a 15° angle. While measuring, the EZ-Sizer tool allows you to simulate the 15° angle with the BT Holder.
- 3) Depending on length of finger it may also be important to shorten the Swan Neck Splint by bending the rings together a little. Remember when squeezing the rings together for the Swan Neck Splint it will start to loosen the fit of that splint.

