



To measure for two swan neck splints worn on the same finger, you need to make a slight modification when measuring.

1. First, measure each splint individually without worrying about the overlap.
2. Then to allow for the overlap, increase the size of the distal ring of the PIP splint by one full size. For example, if the distal ring size was a 7.0, change it to an 8.0. Only change the size of this ring. Everything else stays the same.
3. Using both of the SN holders provided with your kit, you can try two splints on at once to get an idea of how the splints will fit together on the finger. Remember that the plastic rings can misshape because they are plastic. Silver ring splints will hold their shape, be strong and serve as a more universal hyperextension block