



SIRIS BUDDY RING SPLINT (BR)

Adduct Fingers (full offset), Abduct Fingers (no offset)

- 1 Choose Round Ringsizers that fit at the base of the fingers (MP crease) which are being splinted.





- 2 Specify band width: 3, 4 or 6mm.



- 3 Specify: no offset, half offset or full offset.

Note: The combination of band width and offset will determine the "total length" and weight of the splint. Make sure the "total length" does not limit flexion at the adjacent joint.

(Actual size shown)

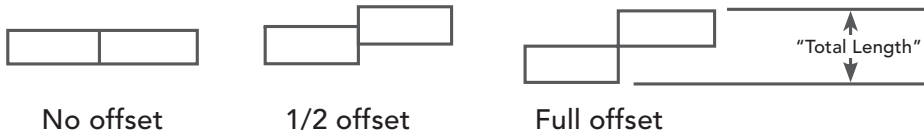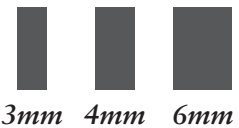

SIRIS™ MEASURING INSTRUCTIONS

SILVER RING SPLINT COMPANY | PHONE: 434.971.4052 or 800.311.7028 | FAX: 434.971.8828 or 888.456.8828

CustomerService@SilverRingSplint.com | www.SilverRingSplint.com