




# SIRIS DORSAL EXTENSION SPLINT (DOE)


*Flexion or Deviation greater than 25°*

## SPECIAL ORDER SPLINT


- 1 Choose Round Ringsizers that, when positioned straight, fit proximal and distal to the joint at the mid-phalanx.

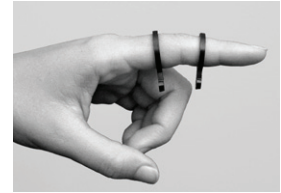


- 2 Measure the lengths from the proximal edge of proximal Ringsizer to distal edge of the distal Ringsizer on both dorsal and volar surfaces.

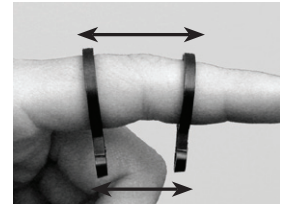

- 3 Measure the length from the proximal edge of proximal Ringsizer to the center of the joint.

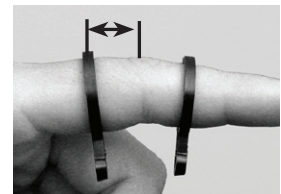

- 4 Measure degree of contracture.

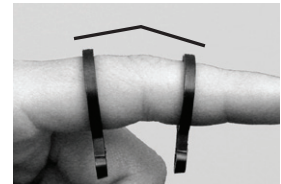

- 5 Specify band width: 3 or 4mm.

*(Actual size shown)*



*Deviation Deformity with bar worn on the side of the joint (shown left)*



*Flexion Deformity with bar worn on top of the joint (shown right)*



## SPLINT ADJUSTMENTS AND FITTING:

- Splint can be adjusted to accommodate a contracture by bending the bar which connects the two rings.

### SIRIS™ MEASURING INSTRUCTIONS

SILVER RING SPLINT COMPANY | PHONE: 434.971.4052 or 800.311.7028 | FAX: 434.971.8828 or 888.456.8828

*CustomerService@SilverRingSplint.com | www.SilverRingSplint.com*