




SIRIS TRIGGER SPLINT (TR)

Recurring Flexor Tenosynovitis of Index-Little Fingers


- 1 Choose a Round Ringsizer that fits at the base of the finger (MP crease).



- 2 Measure the length from the proximal palmer crease to the base of finger (MP crease).

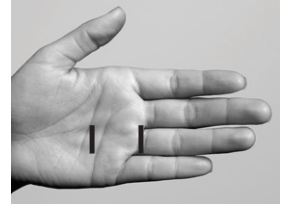

- 3 Measure the length from the base of the finger (MP crease) to within 5 – 8mm proximal to the PIP volar crease.

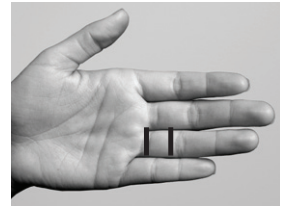

The “dorsal pad” on the splint helps distribute the pressure on the dorsal surface of the finger.



SIRIS™ MEASURING INSTRUCTIONS

SILVER RING SPLINT COMPANY | PHONE: 434.971.4052 or 800.311.7028 | FAX: 434.971.8828 or 888.456.8828

CustomerService@SilverRingSplint.com | www.SilverRingSplint.com